QUIMIURIC – ÁCIDO ÚRICO

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Item Name		#	AUR	
DATA INFORMATION				
UNIT		mg/dL		
DECIMALS		2		
ANALYSIS				
TYPE		END		
Main W. Length 1		505		
Sub W. Length 2		660		
METHOD		Uricase/Peroxidase		
CORRELAÇÃO				
Y =	SLOPE	X +	INTER	
	1		0	
ASPIRATION				
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double		
		VOLUME		
SAMPLE		6		
REAGENT 1		250		
REAGENT 2				
Third Mix.		<input checked="" type="radio"/> OFF <input type="radio"/> ON		
R1 Blank		<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1		
MONITOR				
0 LEVEL POINT		1		
SPAN		3		
PROZONE CHECK				
FIRST	START	END	LIMIT	
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
AUTO RERUN SW				
<input type="radio"/> ON		<input checked="" type="radio"/> OFF		
Auto Rerun Range (Results)				
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
	Lower	Higher		
Serum	0	30		
Urine	*	*		
Plasma	*	*		
CSF	*	*		
Dialysis	*	*		
Other	*	*		
CALIBRATION				
TYPE		Linear		
		STANDARD		
#1	@	#4		
#2		#5		
#3		#6		
NORMAL RANGE				
	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
Serum	3.5	7.2	2.6	6.0
Urine				
Plasma				
CSF				
Dialysis				
Other				
DATA PROCESS				
	START	END		
MAIN	30	31		
SUB				
FACTOR				
BLANK CORRECTION		1		
ABSORBANCE LIMIT				
LOW		-2.500		
HIGH		2.400		
END POINT LIMIT		0.020		
LINEAR CHECK				
Auto Rerun Condition (Absorbance)				
Absorbance Range				
	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Prozone Range		<input type="radio"/> ON <input checked="" type="radio"/> OFF		

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Item Name		#	ADA	
DATA INFORMATION			CALIBRATION	
UNIT		U/L	TYPE	
DECIMALS		2	Linear	
			STANDARD	
			#1	@ #4
			#2	#5
			#3	#6
ANALYSIS				
TYPE		RATE		
Main W. Length 1		546		
Sub W. Length 2		700		
METHOD		ENZ-POD		
CORRELAÇÃO			NORMAL RANGE	
Y =	SLOPE	X +	INTER	
	1		0	
ASPIRATION				
KIND		<input type="radio"/> Single <input checked="" type="radio"/> Double		
		VOLUME		
SAMPLE		4		
REAGENT 1		140		
REAGENT 2		70		
Third Mix.		<input checked="" type="radio"/> OFF <input type="radio"/> ON		
R1 Blank		<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1		
MONITOR			DATA PROCESS	
0 LEVEL POINT		1		
SPAN		3		
PROZONE CHECK				
FIRST	START	END	LIMIT	
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
AUTO RERUN SW				
<input type="radio"/> ON <input checked="" type="radio"/> OFF			Auto Rerun Condition (Absorbance)	
Auto Rerun Range (Results)				
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	Absorbance Range	
	Lower	Higher		
Serum	0	200	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF
Urine	*	*	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF
Plasma	*	*	Prozone Range	
CSF	*	*	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Dialysis	*	*		
Other	*	*		

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Item Name		#	ALB
DATA INFORMATION			
UNIT		g/dL	
DECIMALS		2	
ANALYSIS			
TYPE		END	
Main W. Length 1		600	
Sub W. Length 2		700	
METHOD		Verde de Bromocresol	
CORRELAÇÃO			
Y =	SLOPE	X +	INTER
	1		0
ASPIRATION			
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double	
		VOLUME	
SAMPLE		3	
REAGENT 1		300	
REAGENT 2			
Third Mix.		<input checked="" type="radio"/> OFF <input type="radio"/> ON	
R1 Blank		<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1	
MONITOR			
0 LEVEL POINT		1	
SPAN		3	
PROZONE CHECK			
FIRST	START	END	LIMIT
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW			
<input type="radio"/> ON		<input checked="" type="radio"/> OFF	
Auto Rerun Range (Results)			
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Lower	Higher	
Serum	0	6	
Urine	*	*	
Plasma	*	*	
CSF	*	*	
Dialysis	*	*	
Other	*	*	
CALIBRATION			
TYPE		Linear	
STANDARD			
#1	@	#4	
#2		#5	
#3		#6	
NORMAL RANGE			
	MALE		FEMALE
	LOW	HIGH	LOW HIGH
Serum	3.5	5.0	3.5 5.0
Urine			
Plasma			
CSF			
Dialysis			
Other			
DATA PROCESS			
	START	END	
MAIN	20	22	
SUB			
FACTOR			
BLANK CORRECTION		1	
ABSORBANCE LIMIT			
LOW	-2.500		
HIGH	2.400		
END POINT LIMIT		0.020	
LINEAR CHECK			
Auto Rerun Condition (Absorbance)			
Absorbance Range			
Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Prozone Range		<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	AMI	
DATA INFORMATION			CALIBRATION	
UNIT		U/L	TYPE	
DECIMALS		0	Linear	
			STANDARD	
			#1	#4
			#2	#5
			#3	#6
ANALYSIS				
TYPE		RATE		
Main W. Length 1		405		
Sub W. Length 2		700		
METHOD		Substrato Bloqueado - PNP		
CORRELAÇÃO				
Y =	SLOPE	X +	INTER	
	1		0	
ASPIRATION				
KIND		<input checked="" type="radio"/> Single	<input type="radio"/> Double	
		VOLUME		
SAMPLE		4		
REAGENT 1		200		
REAGENT 2				
Third Mix.		<input checked="" type="radio"/> OFF	<input type="radio"/> ON	
R1 Blank		<input checked="" type="radio"/> Water Blank	<input type="radio"/> R1 Blank 1	
MONITOR				
0 LEVEL POINT		1		
SPAN		3		
PROZONE CHECK				
FIRST	START	END	LIMIT	
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
AUTO RERUN SW				
<input type="radio"/> ON		<input checked="" type="radio"/> OFF		
Auto Rerun Range (Results)				
	<input type="radio"/> ON	<input checked="" type="radio"/> OFF	<input type="radio"/> ON	<input checked="" type="radio"/> OFF
	Lower		Higher	
Serum	2.0		1300	
Urine	*		*	
Plasma	*		*	
CSF	*		*	
Dialysis	*		*	
Other	*		*	
			DATA PROCESS	
			START	END
MAIN			22	26
SUB				
FACTOR				
BLANK CORRECTION			1	
ABSORBANCE LIMIT				
LOW			-2.500	
HIGH			2.400	
END POINT LIMIT				
LINEAR CHECK			40	
Auto Rerun Condition (Absorbance)				
Absorbance Range				
	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Prozone Range			<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	BD
DATA INFORMATION			
UNIT	mg/dL		
DECIMALS	2		
ANALYSIS			
TYPE	END		
Main W. Length 1	546		
Sub W. Length 2	660		
METHOD	Diazo		
CORRELAÇÃO			
Y =	SLOPE	X +	INTER
	1		0
ASPIRATION			
KIND	<input type="radio"/> Single <input checked="" type="radio"/> Double		
	VOLUME		
SAMPLE	25		
REAGENT 1	200		
REAGENT 2	50		
Third Mix.	<input checked="" type="radio"/> OFF <input type="radio"/> ON		
R1 Blank	<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1		
MONITOR			
0 LEVEL POINT	1		
SPAN	3		
PROZONE CHECK			
FIRST	START	END	LIMIT
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW			
<input type="radio"/> ON <input checked="" type="radio"/> OFF			
Auto Rerun Range (Results)			
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Lower	Higher	
Serum	0	20	
Urine	*	*	
Plasma	*	*	
CSF	*	*	
Dialysis	*	*	
Other	*	*	
CALIBRATION			
TYPE		Linear	
STANDARD			
#1	@	#4	
#2		#5	
#3		#6	
NORMAL RANGE			
	MALE		FEMALE
	LOW	HIGH	LOW HIGH
Serum	0.0	0.2	0.0 0.2
Urine			
Plasma			
CSF			
Dialysis			
Other			
DATA PROCESS			
	START	END	
MAIN	53	54	
SUB	30	31	
FACTOR			
BLANK CORRECTION		1	
ABSORBANCE LIMIT			
LOW	-0.1		
HIGH	2.5		
END POINT LIMIT		0.050	
LINEAR CHECK		0	
Auto Rerun Condition (Absorbance)			
Absorbance Range		Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF
		Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF
Prozone Range		<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name	#	BT
DATA INFORMATION		
UNIT	mg/dL	
DECIMALS	2	
ANALYSIS		
TYPE	END	
Main W. Length 1	546	
Sub W. Length 2	660	
METHOD	Diazo	
CORRELAÇÃO		
Y =	SLOPE	X + INTER
	1	0
ASPIRATION		
KIND	<input type="radio"/> Single	<input checked="" type="radio"/> Double
SAMPLE	15	
REAGENT 1	200	
REAGENT 2	120	
Third Mix.	<input checked="" type="radio"/> OFF	<input type="radio"/> ON
R1 Blank	<input checked="" type="radio"/> Water Blank	<input type="radio"/> R1 Blank 1
MONITOR		
0 LEVEL POINT	1	
SPAN	3	
PROZONE CHECK		
FIRST	START	END LIMIT
SECOND		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW		
<input type="radio"/> ON	<input checked="" type="radio"/> OFF	
Auto Rerun Range (Results)		
<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Lower	Higher	
Serum	0 20	
Urine	* *	
Plasma	* *	
CSF	* *	
Dialysis	* *	
Other	* *	
CALIBRATION		
TYPE	Linear	
STANDARD		
#1	@	#4
#2		#5
#3		#6
NORMAL RANGE		
	MALE	FEMALE
	LOW HIGH	LOW HIGH
Serum	0.2 1.0	0.2 1.0
Urine		
Plasma		
CSF		
Dialysis		
Other		
DATA PROCESS		
	START	END
MAIN	53	54
SUB	30	31
FACTOR		
BLANK CORRECTION		1
ABSORBANCE LIMIT		
LOW	-0.1	
HIGH	2.5	
END POINT LIMIT		
		0.050
LINEAR CHECK		
		0
Auto Rerun Condition (Absorbance)		
Absorbance Range		
Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Prozone Range		
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	CAL				
<div>DATA INFORMATION</div>				<div>CALIBRATION</div>			
UNIT		mg/dL		TYPE		Linear	
DECIMALS		2		STANDARD			
				#1	@	#4	
				#2		#5	
				#3		#6	
<div>ANALYSIS</div>				<div>NORMAL RANGE</div>			
TYPE		END		MALE		FEMALE	
Main W. Length 1		660		LOW	HIGH	LOW	HIGH
Sub W. Length 2							
METHOD		Arsenazo III					
<div>CORRELAÇÃO</div>				Serum			
Y =	SLOPE	X +	INTER	Urine	8.5	10.4	8.5
	1		0	Plasma			
				CSF			
<div>ASPIRATION</div>				Dialysis			
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double		Other			
		VOLUME					
SAMPLE		6					
REAGENT 1		300					
REAGENT 2							
Third Mix.	<input checked="" type="radio"/> OFF <input type="radio"/> ON						
R1 Blank	<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1						
<div>MONITOR</div>				<div>DATA PROCESS</div>			
0 LEVEL POINT		1		START	END		
SPAN		3		MAIN	32	33	
				SUB			
				FACTOR			
				BLANK CORRECTION	1		
<div>PROZONE CHECK</div>				<div>ABSORBANCE LIMIT</div>			
FIRST	START	END	LIMIT	LOW	-2.500		
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	HIGH	2.400		
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH				
<div>AUTO RERUN SW</div>				<div>END POINT LIMIT</div>			
<input type="radio"/> ON <input checked="" type="radio"/> OFF				0.020			
<div>Auto Rerun Range (Results)</div>				<div>LINEAR CHECK</div>			
<input type="radio"/> ON <input checked="" type="radio"/> OFF		<input type="radio"/> ON <input checked="" type="radio"/> OFF					
Lower		Higher		<div>Auto Rerun Condition (Absorbance)</div>			
Serum	0	15		Absorbance Range			
Urine	*	*		Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Plasma	*	*		Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
CSF	*	*		<div>Prozone Range</div>			
Dialysis	*	*		<input type="radio"/> ON <input checked="" type="radio"/> OFF			
Other	*	*					

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Item Name	#	CKMB
DATA INFORMATION		CALIBRATION
UNIT	U/L	TYPE
DECIMALS	0	9360
ANALYSIS		
TYPE	RATE	
Main W. Length 1	340	
Sub W. Length 2	700	
METHOD	UV	
CORRELAÇÃO		
Y =	SLOPE	X +
	1	0
ASPIRATION		
KIND	Single	Double
	VOLUME	
SAMPLE	8	
REAGENT 1	200	
REAGENT 2		
Third Mix.	OFF	ON
R1 Blank	Water Blank	R1 Blank 1
MONITOR		
0 LEVEL POINT	1	
SPAN	3	
PROZONE CHECK		
FIRST	START	END
SECOND		LIMIT
HIGHTHIRD		LOW HIGH
		LOW HIGH
AUTO RERUN SW		
	ON	OFF
Auto Rerun Range (Results)		
	ON	OFF
	ON	OFF
	Lower	Higher
Serum	3	1000
Urine	*	*
Plasma	*	*
CSF	*	*
Dialysis	*	*
Other	*	*
		Auto Rerun Condition (Absorbance)
		Absorbance Range
	Lower	ON OFF
	Higher	ON OFF
		Prozone Range
		ON OFF

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Item Name	#	CKNAC
DATA INFORMATION		
UNIT	U/L	
DECIMALS	0	
ANALYSIS		
TYPE	RATE	
Main W. Length 1	340	
Sub W. Length 2	700	
METHOD	UV	
CORRELAÇÃO		
Y =	SLOPE	X + INTER
	1	0
ASPIRATION		
KIND	<input checked="" type="radio"/> Single <input type="radio"/> Double	
SAMPLE	VOLUME	
REAGENT 1	8	
REAGENT 2	200	
Third Mix.	<input checked="" type="radio"/> OFF <input type="radio"/> ON	
R1 Blank	<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1	
MONITOR		
0 LEVEL POINT	1	
SPAN	3	
PROZONE CHECK		
FIRST	START	END LIMIT
SECOND		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW		
<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Auto Rerun Range (Results)		
<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Lower	Higher	
Serum	0 1300	
Urine	* *	
Plasma	* *	
CSF	* *	
Dialysis	* *	
Other	* *	
CALIBRATION		
TYPE	Linear	
STANDARD		
#1	@	#4
#2		#5
#3		#6
NORMAL RANGE		
	MALE	FEMALE
	LOW HIGH	LOW HIGH
Serum	38 174	26 140
Urine		
Plasma		
CSF		
Dialysis		
Other		
DATA PROCESS		
	START	END
MAIN	20	28
SUB		
FACTOR		
BLANK CORRECTION		1
ABSORBANCE LIMIT		
LOW	-2.500	
HIGH	2.400	
END POINT LIMIT		
LINEAR CHECK		40
Auto Rerun Condition (Absorbance)		
Absorbance Range		
Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Prozone Range		
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	CLO
DATA INFORMATION			
UNIT	mmol/L		
DECIMALS	0		
ANALYSIS			
TYPE	END		
Main W. Length 1	505		
Sub W. Length 2	660		
METHOD	Tiocianato de Mercurio		
CORRELAÇÃO			
Y =	SLOPE	X +	INTER
	1		0
ASPIRATION			
KIND	<input checked="" type="radio"/> Single <input type="radio"/> Double		
	VOLUME		
SAMPLE	3		
REAGENT 1	300		
REAGENT 2			
Third Mix.	<input checked="" type="radio"/> OFF	<input type="radio"/> ON	
R1 Blank	<input checked="" type="radio"/> Water Blank	<input type="radio"/> R1 Blank 1	
MONITOR			
0 LEVEL POINT	1		
SPAN	3		
PROZONE CHECK			
FIRST	START	END	LIMIT
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW			
<input type="radio"/> ON	<input checked="" type="radio"/> OFF		
Auto Rerun Range (Results)			
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Lower	Higher	
Serum	80	120	
Urine	*	*	
Plasma	*	*	
CSF	*	*	
Dialysis	*	*	
Other	*	*	
CALIBRATION			
TYPE	Linear		
STANDARD			
#1	@	#4	
#2		#5	
#3		#6	
NORMAL RANGE			
	MALE		FEMALE
	LOW	HIGH	LOW HIGH
Serum	98	106	98 106
Urine			
Plasma			
CSF			
Dialysis			
Other			
DATA PROCESS			
	START	END	
MAIN	53	54	
SUB			
FACTOR			
BLANK CORRECTION		1	
ABSORBANCE LIMIT			
LOW	-2.500		
HIGH	2.400		
END POINT LIMIT			
		0.020	
LINEAR CHECK			
Auto Rerun Condition (Absorbance)			
Absorbance Range			
	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Prozone Range			
		<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	COL
DATA INFORMATION			
UNIT	mg/dL		
DECIMALS	0		
ANALYSIS			
TYPE	END		
Main W. Length 1	505		
Sub W. Length 2	660		
METHOD	Esterase Peroxidase		
CORRELAÇÃO			
Y =	SLOPE	X +	INTER
	1		0
ASPIRATION			
KIND	<input checked="" type="radio"/> Single <input type="radio"/> Double		
	VOLUME		
SAMPLE	3		
REAGENT 1	300		
REAGENT 2			
Third Mix.	<input checked="" type="radio"/> OFF <input type="radio"/> ON		
R1 Blank	<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1		
MONITOR			
0 LEVEL POINT	1		
SPAN	3		
PROZONE CHECK			
FIRST	START	END	LIMIT
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW			
<input type="radio"/> ON <input checked="" type="radio"/> OFF			
Auto Rerun Range (Results)			
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Lower	Higher	
Serum	3	750	
Urine	*	*	
Plasma	*	*	
CSF	*	*	
Dialysis	*	*	
Other	*	*	
CALIBRATION			
TYPE		Linear	
STANDARD			
#1	@	#4	
#2		#5	
#3		#6	
NORMAL RANGE			
	MALE		FEMALE
	LOW	HIGH	LOW HIGH
Serum	140	200	140 200
Urine			
Plasma			
CSF			
Dialysis			
Other			
DATA PROCESS			
	START	END	
MAIN	33	34	
SUB			
FACTOR			
BLANK CORRECTION		1	
ABSORBANCE LIMIT			
LOW	-2.500		
HIGH	2.400		
END POINT LIMIT		0.020	
LINEAR CHECK			
Auto Rerun Condition (Absorbance)			
Absorbance Range			
	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Prozone Range		<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	CREA	
DATA INFORMATION				
UNIT		mg/dL		
DECIMALS		2		
ANALYSIS				
TYPE		RATE		
Main W. Length 1		505		
Sub W. Length 2		570		
METHOD		Jaffé Modificado		
CORRELAÇÃO				
Y =	SLOPE	X +	INTER	
	1		0	
ASPIRATION				
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double		
		VOLUME		
SAMPLE		20		
REAGENT 1		200		
REAGENT 2				
Third Mix.		<input checked="" type="radio"/> OFF <input type="radio"/> ON		
R1 Blank		<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1		
MONITOR				
0 LEVEL POINT		1		
SPAN		3		
PROZONE CHECK				
FIRST	START	END	LIMIT	
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
AUTO RERUN SW				
<input type="radio"/> ON		<input checked="" type="radio"/> OFF		
Auto Rerun Range (Results)				
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
	Lower	Higher		
Serum	0	20		
Urine	*	*		
Plasma	*	*		
CSF	*	*		
Dialysis	*	*		
Other	*	*		
CALIBRATION				
TYPE		Linear		
STANDARD				
#1	@	#4		
#2		#5		
#3		#6		
NORMAL RANGE				
	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
Serum	0.7	1.4	0.5	1.3
Urine				
Plasma				
CSF				
Dialysis				
Other				
DATA PROCESS				
	START	END		
MAIN	16	28		
SUB				
FACTOR				
BLANK CORRECTION		1		
ABSORBANCE LIMIT				
LOW		-2.500		
HIGH		2.400		
END POINT LIMIT				
LINEAR CHECK		90		
Auto Rerun Condition (Absorbance)				
Absorbance Range				
	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Prozone Range		<input type="radio"/> ON <input checked="" type="radio"/> OFF		

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Item Name	#	FER
DATA INFORMATION		CALIBRATION
UNIT	µg/dL	TYPE
DECIMALS	0	Linear
ANALYSIS		STANDARD
TYPE	END	#1 @ #4
Main W. Length 1	570	#2 #5
Sub W. Length 2		#3 #6
METHOD	Ferrozine	
CORRELAÇÃO		NORMAL RANGE
Y =	SLOPE	X +
	1	0
ASPIRATION		
KIND	<input type="radio"/> Single <input checked="" type="radio"/> Double	
SAMPLE	VOLUME	
REAGENT 1	30	
REAGENT 2	120	
	30	
Third Mix.	<input checked="" type="radio"/> OFF <input type="radio"/> ON	
R1 Blank	<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1	
MONITOR		
0 LEVEL POINT	1	
SPAN	3	
PROZONE CHECK		
FIRST	START	END
SECOND		LIMIT
HIGHTHIRD		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW		
<input type="radio"/> ON	<input checked="" type="radio"/> OFF	
Auto Rerun Range (Results)		
<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Lower	Higher	
Serum	6 1000	
Urine	* *	
Plasma	* *	
CSF	* *	
Dialysis	* *	
Other	* *	
		Auto Rerun Condition (Absorbance)
		Absorbance Range
		Lower <input type="radio"/> ON <input checked="" type="radio"/> OFF
		Higher <input type="radio"/> ON <input checked="" type="radio"/> OFF
		Prozone Range <input type="radio"/> ON <input checked="" type="radio"/> OFF

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Item Name	#	FAL
DATA INFORMATION		
UNIT	U/L	
DECIMALS	0	
ANALYSIS		
TYPE	RATE	
Main W. Length 1	405	
Sub W. Length 2	700	
METHOD	p-Nitrofenilfosfato (IFCC)	
CORRELAÇÃO		
Y =	SLOPE	X + INTER
	1	0
ASPIRATION		
KIND	<input type="radio"/> Single	<input checked="" type="radio"/> Double
SAMPLE	5	
REAGENT 1	250	
REAGENT 2	60	
Third Mix.	<input checked="" type="radio"/> OFF	<input type="radio"/> ON
R1 Blank	<input checked="" type="radio"/> Water Blank	<input type="radio"/> R1 Blank 1
MONITOR		
0 LEVEL POINT	1	
SPAN	3	
PROZONE CHECK		
FIRST	START	END LIMIT
SECOND		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW		
<input type="radio"/> ON	<input checked="" type="radio"/> OFF	
Auto Rerun Range (Results)		
<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Lower	Higher	
Serum	1	1200
Urine	*	*
Plasma	*	*
CSF	*	*
Dialysis	*	*
Other	*	*
CALIBRATION		
TYPE	Linear	
STANDARD		
#1	@	#4
#2		#5
#3		#6
NORMAL RANGE		
	MALE	FEMALE
	LOW	HIGH
Serum	53	128
Urine		42
Plasma		98
CSF		
Dialysis		
Other		
DATA PROCESS		
	START	END
MAIN	37	41
SUB		
FACTOR		
BLANK CORRECTION		1
ABSORBANCE LIMIT		
LOW	-2.500	
HIGH	2.400	
END POINT LIMIT		
LINEAR CHECK		40
Auto Rerun Condition (Absorbance)		
Absorbance Range		
Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Prozone Range		
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	FOS				
DATA INFORMATION				CALIBRATION			
UNIT		mg/dL		TYPE		Linear	
DECIMALS		2		STANDARD			
				#1	@	#4	
				#2		#5	
				#3		#6	
ANALYSIS				NORMAL RANGE			
TYPE		END					
Main W. Length 1		340					
Sub W. Length 2		660					
METHOD		Molibdato-UV					
CORRELAÇÃO							
Y =	SLOPE	X +	INTER				
	1		0				
ASPIRATION							
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double					
		VOLUME					
SAMPLE		4					
REAGENT 1		200					
REAGENT 2							
Third Mix.		<input checked="" type="radio"/> OFF <input type="radio"/> ON					
R1 Blank		<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1					
MONITOR							
0 LEVEL POINT		1					
SPAN		3					
PROZONE CHECK							
FIRST	START	END	LIMIT				
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH				
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH				
AUTO RERUN SW							
<input type="radio"/> ON		<input checked="" type="radio"/> OFF					
Auto Rerun Range (Results)				Auto Rerun Condition (Absorbance)			
<input type="radio"/> ON <input checked="" type="radio"/> OFF		<input type="radio"/> ON <input checked="" type="radio"/> OFF					
Lower		Higher		Absorbance Range			
Serum	0	15		Lower		<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Urine	*	*		Higher		<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Plasma	*	*					
CSF	*	*		Prozone Range			
Dialysis	*	*		<input type="radio"/> ON <input checked="" type="radio"/> OFF			
Other	*	*					

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Item Name	#	GGT
DATA INFORMATION		CALIBRATION
UNIT	U/L	TYPE
DECIMALS	0	Linear
ANALYSIS		STANDARD
TYPE	RATE	#1 @ #4
Main W. Length 1	405	#2 #5
Sub W. Length 2	700	#3 #6
METHOD	IFCC-Cinético	NORMAL RANGE
CORRELAÇÃO		MALE FEMALE
Y =	SLOPE X + INTER	LOW HIGH LOW HIGH
	1 0	Serum 12 64 9 36
ASPIRATION		Urine
KIND	Single Double	Plasma
	VOLUME	CSF
SAMPLE	9	Dialysis
REAGENT 1	200	Other
REAGENT 2		DATA PROCESS
Third Mix.	OFF ON	START END
R1 Blank	Water Blank R1 Blank 1	MAIN 19 29
MONITOR		SUB
0 LEVEL POINT	1	FACTOR
SPAN	3	BLANK CORRECTION 1
PROZONE CHECK		ABSORBANCE LIMIT
FIRST	START END LIMIT	LOW -2.500
SECOND	LOW HIGH	HIGH 2.400
HIGHTHIRD	LOW HIGH	END POINT LIMIT
AUTO RERUN SW		LINEAR CHECK 40
ON OFF		Auto Rerun Condition (Absorbance)
Auto Rerun Range (Results)		Absorbance Range
ON OFF ON OFF	Lower Higher	Lower ON OFF
		Higher ON OFF
Serum	6 800	Prozone Range ON OFF
Urine	* *	
Plasma	* *	
CSF	* *	
Dialysis	* *	
Other	* *	

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Item Name	#	GLI
DATA INFORMATION		
UNIT	mg/dL	
DECIMALS	0	
ANALYSIS		
TYPE	END	
Main W. Length 1	505	
Sub W. Length 2	660	
METHOD	Oxidase	
CORRELAÇÃO		
Y =	SLOPE	X + INTER
	1	0
ASPIRATION		
KIND	<input checked="" type="radio"/> Single	<input type="radio"/> Double
	VOLUME	
SAMPLE	3	
REAGENT 1	300	
REAGENT 2		
Third Mix.	<input checked="" type="radio"/> OFF	<input type="radio"/> ON
R1 Blank	<input checked="" type="radio"/> Water Blank	<input type="radio"/> R1 Blank 1
MONITOR		
0 LEVEL POINT	1	
SPAN	3	
PROZONE CHECK		
FIRST	START	END LIMIT
SECOND		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW		
<input type="radio"/> ON	<input checked="" type="radio"/> OFF	
Auto Rerun Range (Results)		
<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Lower	Higher
Serum	0	400
Urine	*	*
Plasma	*	*
CSF	*	*
Dialysis	*	*
Other	*	*
CALIBRATION		
TYPE	Linear	
STANDARD		
#1	@	#4
#2		#5
#3		#6
NORMAL RANGE		
	MALE	FEMALE
	LOW	HIGH
Serum	70	99
Urine		
Plasma		
CSF		
Dialysis		
Other		
DATA PROCESS		
	START	END
MAIN	56	54
SUB		
FACTOR		
BLANK CORRECTION		1
ABSORBANCE LIMIT		
LOW	-2.500	
HIGH	2.400	
END POINT LIMIT		
		0.020
LINEAR CHECK		
Auto Rerun Condition (Absorbance)		
Absorbance Range		
	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF
	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF
Prozone Range		
		<input type="radio"/> ON <input checked="" type="radio"/> OFF

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Item Name	#	HDL
DATA INFORMATION		
UNIT	mg/dL	
DECIMALS	0	
ANALYSIS		
TYPE	END	
Main W. Length 1	600	
Sub W. Length 2	700	
METHOD	Direto	
CORRELAÇÃO		
Y =	SLOPE	X + INTER
	1	0
ASPIRATION		
KIND	<input type="radio"/> Single	<input checked="" type="radio"/> Double
SAMPLE	2.5	
REAGENT 1	195	
REAGENT 2	65	
Third Mix.	<input checked="" type="radio"/> OFF	<input type="radio"/> ON
R1 Blank	<input checked="" type="radio"/> Water Blank	<input type="radio"/> R1 Blank 1
MONITOR		
0 LEVEL POINT	1	
SPAN	3	
PROZONE CHECK		
FIRST	START	END LIMIT
SECOND		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD		<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW		
<input type="radio"/> ON	<input checked="" type="radio"/> OFF	
Auto Rerun Range (Results)		
<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Lower	Higher	
Serum	1	100
Urine	*	*
Plasma	*	*
CSF	*	*
Dialysis	*	*
Other	*	*
CALIBRATION		
TYPE	Linear	
STANDARD		
#1	@	#4
#2		#5
#3		#6
NORMAL RANGE		
	MALE	FEMALE
	LOW	HIGH
Serum	30	70
Urine		
Plasma		
CSF		
Dialysis		
Other		
DATA PROCESS		
	START	END
MAIN	53	54
SUB	30	31
FACTOR		
BLANK CORRECTION		1
ABSORBANCE LIMIT		
LOW	-0.100	
HIGH	0.500	
END POINT LIMIT		
LINEAR CHECK		0.020
Auto Rerun Condition (Absorbance)		
Absorbance Range		
Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
Prozone Range		
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	DHL
DATA INFORMATION			
UNIT		U/L	
DECIMALS		0	
ANALYSIS			
TYPE		RATE	
Main W. Length 1		340	
Sub W. Length 2			
METHOD		Piruvato	
CORRELAÇÃO			
Y =	SLOPE	X +	INTER
	1		0
ASPIRATION			
KIND		<input checked="" type="radio"/> Single	<input type="radio"/> Double
		VOLUME	
SAMPLE		5	
REAGENT 1		200	
REAGENT 2			
Third Mix.	<input checked="" type="radio"/> OFF	<input type="radio"/> ON	
R1 Blank	<input checked="" type="radio"/> Water Blank	<input type="radio"/> R1 Blank 1	
MONITOR			
0 LEVEL POINT		1	
SPAN		3	
PROZONE CHECK			
FIRST	START	END	LIMIT
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW			
<input type="radio"/> ON		<input checked="" type="radio"/> OFF	
Auto Rerun Range (Results)			
	<input type="radio"/> ON	<input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF
	Lower	Higher	
Serum	4.7	1250	
Urine	*	*	
Plasma	*	*	
CSF	*	*	
Dialysis	*	*	
Other	*	*	
CALIBRATION			
TYPE		Linear	
STANDARD			
#1	@	#4	
#2		#5	
#3		#6	
NORMAL RANGE			
	MALE		FEMALE
	LOW	HIGH	LOW
	(*)	(*)	(*)
Serum			
Urine			
Plasma			
CSF			
Dialysis			
Other			
DATA PROCESS			
	START	END	
MAIN	19	29	
SUB			
FACTOR			
BLANK CORRECTION		1	
ABSORBANCE LIMIT			
LOW		-2.500	
HIGH		2.400	
END POINT LIMIT			
LINEAR CHECK		40	
Auto Rerun Condition (Absorbance)			
Absorbance Range			
Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Prozone Range		<input type="radio"/> ON <input checked="" type="radio"/> OFF	

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Item Name		#	LACTE	
DATA INFORMATION			CALIBRATION	
UNIT		mg/dL	TYPE	
DECIMALS		1	Linear	
			STANDARD	
			#1	@ #4
			#2	#5
			#3	#6
ANALYSIS			NORMAL RANGE	
TYPE		END		
Main W. Length 1		546		
Sub W. Length 2				
METHOD		Colorimétrico enzimático		
CORRELAÇÃO				
Y =	SLOPE	X +	INTER	
	1		0	
ASPIRATION				
KIND		● Single ○ Double		
		VOLUME		
SAMPLE		3		
REAGENT 1		300		
REAGENT 2				
Third Mix.		● OFF ○ ON		
R1 Blank		● Water Blank ○ R1 Blank 1		
MONITOR				
0 LEVEL POINT		1		
SPAN		3		
PROZONE CHECK				
FIRST	START	END	LIMIT	
SECOND			● LOW ○ HIGH	
HIGHTHIRD			● LOW ○ HIGH	
AUTO RERUN SW				
○ ON		● OFF		
Auto Rerun Range (Results)			Auto Rerun Condition (Absorbance)	
○ ON ● OFF		○ ON ● OFF		
Lower		Higher		
Serum	*	*		
Urine	0.3	90		
Plasma	*	*		
CSF	0.3	90		
Dialysis	*	*		
Other	*	*		
Absorbance Range		Lower ○ ON ● OFF		
		Higher ○ ON ● OFF		
Prozone Range		○ ON ● OFF		

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Item Name		#	MAG	
DATA INFORMATION			CALIBRATION	
UNIT		mg/dL	TYPE	
DECIMALS		2	Linear	
			STANDARD	
			#1	@ #4
			#2	#5
			#3	#6
ANALYSIS			NORMAL RANGE	
TYPE		END		
Main W. Length 1		570		
Sub W. Length 2		660		
METHOD		Arsenazo		
CORRELAÇÃO				
Y =	SLOPE	X +	INTER	
	1		0	
ASPIRATION				
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double		
		VOLUME		
SAMPLE		4		
REAGENT 1		200		
REAGENT 2				
Third Mix.		<input checked="" type="radio"/> OFF <input type="radio"/> ON		
R1 Blank		<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1		
MONITOR				
0 LEVEL POINT		1		
SPAN		3		
PROZONE CHECK				
FIRST	START	END	LIMIT	
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
AUTO RERUN SW				
<input type="radio"/> ON		<input checked="" type="radio"/> OFF		
Auto Rerun Range (Results)			Auto Rerun Condition (Absorbance)	
<input type="radio"/> ON <input checked="" type="radio"/> OFF		<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Lower		Higher		
Serum	0.12	6.10		
Urine	*	*		
Plasma	*	*		
CSF	*	*		
Dialysis	*	*		
Other	*	*		
Absorbance Range		Prozone Range		
Lower		<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Higher		<input type="radio"/> ON <input checked="" type="radio"/> OFF		

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Item Name		#	PROT							
<div>DATA INFORMATION</div>					<div>CALIBRATION</div>					
UNIT		g/dL			TYPE		Linear			
DECIMALS		2			STANDARD					
					#1	@	#4			
					#2		#5			
					#3		#6			
<div>ANALYSIS</div>					<div>NORMAL RANGE</div>					
TYPE		END					MALE		FEMALE	
Main W. Length 1		546					LOW	HIGH	LOW	HIGH
Sub W. Length 2		700								
METHOD		Biureto								
<div>CORRELAÇÃO</div>					Serum					
Y =	SLOPE	X +	INTER		Urine					
	1		0		Plasma					
					CSF					
<div>ASPIRATION</div>					Dialysis					
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double			Other					
		VOLUME								
SAMPLE		4			<div>DATA PROCESS</div>					
REAGENT 1		200			START		END			
REAGENT 2					MAIN		53 54			
					SUB					
Third Mix. <input checked="" type="radio"/> OFF <input type="radio"/> ON					FACTOR					
R1 Blank <input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1					BLANK CORRECTION					
					1					
<div>MONITOR</div>					ABSORBANCE LIMIT					
0 LEVEL POINT		1			LOW		-2.500			
SPAN		3			HIGH		2.400			
<div>PROZONE CHECK</div>					END POINT LIMIT					
FIRST	START	END	LIMIT	<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	0.020					
SECOND				<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	LINEAR CHECK					
HIGHTHIRD										
<div>AUTO RERUN SW</div>					<div>Auto Rerun Condition (Absorbance)</div>					
<input type="radio"/> ON <input checked="" type="radio"/> OFF					Absorbance Range					
<div>Auto Rerun Range (Results)</div>					Lower <input type="radio"/> ON <input checked="" type="radio"/> OFF					
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF			Higher <input type="radio"/> ON <input checked="" type="radio"/> OFF					
Serum	0.1	15.0			Lower		<input type="radio"/> ON <input checked="" type="radio"/> OFF			
Urine	*	*			Higher		<input type="radio"/> ON <input checked="" type="radio"/> OFF			
Plasma	*	*			Prozone Range					
CSF	*	*			<input type="radio"/> ON <input checked="" type="radio"/> OFF					
Dialysis	*	*								
Other	*	*								

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Item Name		#	TGO							
<div>DATA INFORMATION</div>					<div>CALIBRATION</div>					
UNIT		U/L		TYPE		Linear				
DECIMALS		0		STANDARD						
				#1 @ #4						
				#2 #5						
				#3 #6						
<div>ANALYSIS</div>					<div>NORMAL RANGE</div>					
TYPE		RATE				MALE		FEMALE		
Main W. Length 1		340				LOW HIGH		LOW HIGH		
Sub W. Length 2		700								
METHOD		UV								
<div>CORRELAÇÃO</div>					<div>Serum</div>					
Y =	SLOPE	X +	INTER		<div>Urine</div>					
	1		0		<div>Plasma</div>					
					<div>CSF</div>					
<div>ASPIRATION</div>					<div>Dialysis</div>					
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double				<div>Other</div>				
		VOLUME								
SAMPLE		20				<div>DATA PROCESS</div>				
REAGENT 1		200				START		END		
REAGENT 2						MAIN		22 26		
						SUB				
Third Mix. <input checked="" type="radio"/> OFF <input type="radio"/> ON						FACTOR				
R1 Blank <input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1						BLANK CORRECTION		1		
<div>MONITOR</div>					<div>ABSORBANCE LIMIT</div>					
0 LEVEL POINT		1				LOW		-1.000		
SPAN		3				HIGH		2.400		
<div>PROZONE CHECK</div>					<div>END POINT LIMIT</div>					
FIRST	START	END	LIMIT	<input checked="" type="radio"/> LOW <input type="radio"/> HIGH		LINEAR CHECK		40		
SECOND				<input checked="" type="radio"/> LOW <input type="radio"/> HIGH						
HIGHTHIRD										
<div>AUTO RERUN SW</div>					<div>Auto Rerun Condition (Absorbance)</div>					
<input type="radio"/> ON <input checked="" type="radio"/> OFF						Absorbance Range				
Auto Rerun Range (Results)						Lower		<input type="radio"/> ON <input checked="" type="radio"/> OFF		
		<input type="radio"/> ON <input checked="" type="radio"/> OFF		<input type="radio"/> ON <input checked="" type="radio"/> OFF		Higher		<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Serum	Lower	0		Higher		400				
Urine		*				*				
Plasma		*				*				
CSF		*				*				
Dialysis		*				*				
Other		*				*				
					<div>Prozone Range</div>					
					<input type="radio"/> ON <input checked="" type="radio"/> OFF					

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Item Name		#	TGP
DATA INFORMATION			
UNIT	U/L		
DECIMALS	0		
ANALYSIS			
TYPE	RATE		
Main W. Length 1	340		
Sub W. Length 2	700		
METHOD	UV		
CORRELAÇÃO			
Y =	SLOPE	X +	INTER
	1		0
ASPIRATION			
KIND	<input checked="" type="radio"/> Single <input type="radio"/> Double		
	VOLUME		
SAMPLE	20		
REAGENT 1	200		
REAGENT 2			
Third Mix.	<input checked="" type="radio"/> OFF <input type="radio"/> ON		
R1 Blank	<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1		
MONITOR			
0 LEVEL POINT	1		
SPAN	3		
PROZONE CHECK			
FIRST	START	END	LIMIT
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH
AUTO RERUN SW			
<input type="radio"/> ON <input checked="" type="radio"/> OFF			
Auto Rerun Range (Results)			
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF	
	Lower	Higher	
Serum	0	400	
Urine	*	*	
Plasma	*	*	
CSF	*	*	
Dialysis	*	*	
Other	*	*	
CALIBRATION			
TYPE		Linear	
STANDARD			
#1	@	#4	
#2		#5	
#3		#6	
NORMAL RANGE			
	MALE		FEMALE
	LOW	HIGH	LOW HIGH
Serum	0	36	0 36
Urine			
Plasma			
CSF			
Dialysis			
Other			
DATA PROCESS			
	START	END	
MAIN	22	26	
SUB			
FACTOR			
BLANK CORRECTION		1	
ABSORBANCE LIMIT			
LOW	-1.000		
HIGH	2.400		
END POINT LIMIT			
LINEAR CHECK		40	
Auto Rerun Condition (Absorbance)			
Absorbance Range		Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF
		Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF
Prozone Range			<input type="radio"/> ON <input checked="" type="radio"/> OFF

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Item Name		#	TRI	
DATA INFORMATION				
UNIT		mg/dL		
DECIMALS		0		
ANALYSIS				
TYPE		END		
Main W. Length 1		505		
Sub W. Length 2		660		
METHOD		GPO/Peroxidase		
CORRELAÇÃO				
Y =	SLOPE	X +	INTER	
	1		0	
ASPIRATION				
KIND		<input checked="" type="radio"/> Single <input type="radio"/> Double		
		VOLUME		
SAMPLE		3		
REAGENT 1		300		
REAGENT 2				
Third Mix.	<input checked="" type="radio"/> OFF <input type="radio"/> ON			
R1 Blank	<input checked="" type="radio"/> Water Blank <input type="radio"/> R1 Blank 1			
MONITOR				
0 LEVEL POINT		1		
SPAN		3		
PROZONE CHECK				
FIRST	START	END	LIMIT	
SECOND			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
HIGHTHIRD			<input checked="" type="radio"/> LOW <input type="radio"/> HIGH	
AUTO RERUN SW				
<input type="radio"/> ON <input checked="" type="radio"/> OFF				
Auto Rerun Range (Results)				
	<input type="radio"/> ON <input checked="" type="radio"/> OFF	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
	Lower	Higher		
Serum	2	600		
Urine	*	*		
Plasma	*	*		
CSF	*	*		
Dialysis	*	*		
Other	*	*		
CALIBRATION				
TYPE		Linear		
STANDARD				
#1	@	#4		
#2		#5		
#3		#6		
NORMAL RANGE				
	MALE		FEMALE	
	LOW	HIGH	LOW	HIGH
Serum	35	160	35	160
Urine				
Plasma				
CSF				
Dialysis				
Other				
DATA PROCESS				
	START	END		
MAIN	53	54		
SUB				
FACTOR				
BLANK CORRECTION		1		
ABSORBANCE LIMIT				
LOW		-2.500		
HIGH		2.400		
END POINT LIMIT				
LINEAR CHECK		0.020		
Auto Rerun Condition (Absorbance)				
Absorbance Range				
	Lower	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
	Higher	<input type="radio"/> ON <input checked="" type="radio"/> OFF		
Prozone Range		<input type="radio"/> ON <input checked="" type="radio"/> OFF		

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Item Name	#	URE
DATA INFORMATION		CALIBRATION
UNIT	mg/dL	TYPE
DECIMALS	0	Linear
ANALYSIS		STANDARD
TYPE	END	#1 @ #4
Main W. Length 1	340	#2 #5
Sub W. Length 2	700	#3 #6
METHOD	Urease/GluDH	NORMAL RANGE
CORRELAÇÃO		MALE FEMALE
Y =	SLOPE X + INTER	LOW HIGH LOW HIGH
	1 0	Serum 15 38 15 38
ASPIRATION		Urine
KIND	Single Double	Plasma
	VOLUME	CSF
SAMPLE	3	Dialysis
REAGENT 1	300	Other
REAGENT 2		DATA PROCESS
Third Mix.	OFF ON	START END
R1 Blank	Water Blank R1 Blank 1	MAIN 34 38
MONITOR		SUB
0 LEVEL POINT	1	FACTOR
SPAN	3	BLANK CORRECTION 1
PROZONE CHECK		ABSORBANCE LIMIT
FIRST	START END LIMIT	LOW 0.600
SECOND	LOW HIGH	HIGH 2.400
HIGHTHIRD	LOW HIGH	END POINT LIMIT 0
AUTO RERUN SW		LINEAR CHECK 90
ON OFF		Auto Rerun Condition (Absorbance)
Auto Rerun Range (Results)		Absorbance Range
ON OFF	ON OFF	Lower ON OFF
Lower	Higher	Higher ON OFF
Serum 0 200		Prozone Range ON OFF
Urine *		
Plasma *		
CSF *		
Dialysis *		
Other *		

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