## QUIMIURIC – ÁCIDO ÚRICO

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### PARAMETERS

<b>TEST NAME:</b>	AUR	<b>R1:</b>	200 µL
<b>TEST N°:</b>		<b>R2:</b>	0 µL
<b>FULL NAME:</b>	Ácido Úrico	<b>SAMPLE VOLUME:</b>	5 µL
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---
<b>PRI. WAVE:</b>	510	<b>CONCENTRATION:</b>	0      25
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---
<b>REAC. TIME:</b>	-1      18	<b>FACTOR:</b>	---
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---
<b>UNIT:</b>	mg/dL		
<b>PRECISION:</b>	0.01		

### REFERENCE

N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)

### CALIBRATION RULE

<b>RULE:</b>	ONE POINT LINEAR
<b>SENSITIVITY:</b>	
<b>REPLICATES:</b>	2
<b>INTERVAL:</b>	
<b>DIFFERENCE LIMIT:</b>	
<b>SD:</b>	0
<b>BLANK RESPONSE:</b>	
<b>ERROR LIMIT:</b>	
<b>COEFFICIENT:</b>	
<b>CONCENTRATION:</b>	(#)

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### PARAMETERS

TEST NAME:	ADA	R1:	180 µL
TEST N°:		R2:	90 µL
FULL NAME:	Adenosina Deaminase	SAMPLE VOLUME:	6 µL
STANDARD N°:	(*)	R1 BLANK:	---
ANALY. TYPE:	KINETIC	MIXED REAG BLK:	---
PRI. WAVE:	546	CONCENTRATION:	0      200
SEC. WAVE:	---	LINEARITY LIMIT:	0.2
TREND:	ASCENDING	SUBSTRATE LIMIT:	---
REAC. TIME:	7      18	FACTOR:	---
INCUBA TIME:	16	PROZONE CHECK:	---
UNIT:	U/L		
PRECISION:	0.01		

### REFERENCE

N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
1		SERUM	1 - 100 YEAR	0.00	15.0
2		LIQ PLEURAL	1 - 100 YEAR	0.00	30.0
3		LCR	1 - 100 YEAR	0.00	9.0

### CALIBRATION RULE

RULE:	ONE POINT LINEAR
SENSITIVITY:	
REPLICATES:	2
INTERVAL:	
DIFFERENCE LIMIT:	
SD:	
BLANK RESPONSE:	
ERROR LIMIT:	
COEFFICIENT:	0
CONCENTRATION:	(#)

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	ALB	<b>R1:</b>	300 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Albumina	<b>SAMPLE VOLUME:</b>	3 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	630	<b>CONCENTRATION:</b>	0.2	7.0	
<b>SEC. WAVE:</b>	---	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	g/dL				
<b>PRECISION:</b>	0.01				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	AMI	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Amilase	<b>SAMPLE VOLUME:</b>	5 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINECT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	405	<b>CONCENTRATION:</b>	5	2000	
<b>SEC. WAVE:</b>	---	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	3            13	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	U/L				
<b>PRECISION:</b>	INTEGER				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	BD	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	50 µL		
<b>FULL NAME:</b>	Bili Direta	<b>SAMPLE VOLUME:</b>	25 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	546	<b>CONCENTRATION:</b>	0	20	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1      17	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	16	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	0.01				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	BT	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	120 µL		
<b>FULL NAME:</b>	Bili Total	<b>SAMPLE VOLUME:</b>	12 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	546	<b>CONCENTRATION:</b>	0	20	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1      18	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	0.01				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	SINGLE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>					
<b>CONCENTRATION:</b>	(#)				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	CAL	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Cálcio	<b>SAMPLE VOLUME:</b>	4 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	630	<b>CONCENTRATION:</b>	0.4	15	
<b>SEC. WAVE:</b>	---	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1      10	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	0.01				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	(#)				

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<u>PARAMETERS</u>					
TEST NAME:	CKMB	R1:	200 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	CKMB	SAMPLE VOLUME:	10 µL		
STANDARD N°:		R1 BLANK:	---		
ANALY. TYPE:	KINECT	MIXED REAG BLK:	---		
PRI. WAVE:	340	CONCENTRATION:	3	600	
SEC. WAVE:	670	LINEARITY LIMIT:	0.2000		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	17      34	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	U/L				
PRECISION:	1				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	ONE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:	0				
CONCENTRATION:					

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<u>PARAMETERS</u>					
TEST NAME:	CKNAC	R1:	200 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	CKNAC	SAMPLE VOLUME:	10 µL		
STANDARD N°:		R1 BLANK:	---		
ANALY. TYPE:	KINECT	MIXED REAG BLK:	---		
PRI. WAVE:	340	CONCENTRATION:	9	1500	
SEC. WAVE:	670	LINEARITY LIMIT:	0.2000		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	7            14	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	U/L				
PRECISION:	INTEGER				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	ONE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:	0				
CONCENTRATION:	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	CLO	<b>R1:</b>	300 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	COLORO	<b>SAMPLE VOLUME:</b>	3 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	510	<b>CONCENTRATION:</b>	80	120	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1                  6	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mmol/L				
<b>PRECISION:</b>	INTEGER				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	COL	<b>R1:</b>	300 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Colesterol	<b>SAMPLE VOLUME:</b>	3 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	510	<b>CONCENTRATION:</b>	0	1000	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1            20	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	INTEGER				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	CRE	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Creatinina	<b>SAMPLE VOLUME:</b>	20 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	FIXED TIME	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	510	<b>CONCENTRATION:</b>	0	20	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	2            12	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	0.01				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	FER	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	50 µL		
<b>FULL NAME:</b>	Ferro	<b>SAMPLE VOLUME:</b>	20 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	546	<b>CONCENTRATION:</b>	4	1000	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1      17	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	16	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	ug/dL				
<b>PRECISION:</b>	1				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	FAL	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Fosfatase Alcalina	<b>SAMPLE VOLUME:</b>	4 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINECT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	405	<b>CONCENTRATION:</b>	2	1500	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	3            13	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	U/L				
<b>PRECISION:</b>	1				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	FOS	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Fósforo	<b>SAMPLE VOLUME:</b>	4 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	340	<b>CONCENTRATION:</b>	0	15	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1            17	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	0.01				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	GGT	<b>R1:</b>	180 µL		
<b>TEST N°:</b>		<b>R2:</b>	45 µL		
<b>FULL NAME:</b>	Gama GT	<b>SAMPLE VOLUME:</b>	10 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINECT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	405	<b>CONCENTRATION:</b>	3	1000	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	0.2000		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	3            10	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	16	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	U/L				
<b>PRECISION:</b>	INTEGER				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	GGT	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Gama GT	<b>SAMPLE VOLUME:</b>	9 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINECT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	405	<b>CONCENTRATION:</b>	3	1000	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	0.2000		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	3	10	<b>FACTOR:</b>	---	
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	U/L				
<b>PRECISION:</b>	INTEGER				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	GLI	<b>R1:</b>	300 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Glicose	<b>SAMPLE VOLUME:</b>	3 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	510	<b>CONCENTRATION:</b>	0	500	
<b>SEC. WAVE:</b>	---	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1            17	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	1				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	HDL	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	67 µL		
<b>FULL NAME:</b>	HDL Colesterol	<b>SAMPLE VOLUME:</b>	3 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	578	<b>CONCENTRATION:</b>	0      200		
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1      17	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	16	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	1				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
TEST NAME:	DHL	R1:	200 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	DHL	SAMPLE VOLUME:	5 µL		
STANDARD N°:	(*)	R1 BLANK:	---		
ANALY. TYPE:	KINETIC	MIXED REAG BLK:	---		
PRI. WAVE:	340	CONCENTRATION:	2	1250	
SEC. WAVE:	670	LINEARITY LIMIT:	---		
TREND:	DESCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	3            13	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	U/L				
PRECISION:	1				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	ONE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:					
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:	0				
CONCENTRATION:	#				

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<u>PARAMETERS</u>					
TEST NAME:	LACTE	R1:	300 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	Lactato	SAMPLE VOLUME:	3 µL		
STANDARD N°:	(*)	R1 BLANK:	---		
ANALY. TYPE:	ENDPOINT	MIXED REAG BLK:	---		
PRI. WAVE:	546	CONCENTRATION:	0.3	90.0	
SEC. WAVE:		LINEARITY LIMIT:	---		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	0            17	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	mg/dL				
PRECISION:	0.01				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	ONE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:	0				
CONCENTRATION:	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	LIPAE	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	50 µL		
<b>FULL NAME:</b>	Lipase	<b>SAMPLE VOLUME:</b>	4 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINETIC	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	578	<b>CONCENTRATION:</b>	3	300	
<b>SEC. WAVE:</b>		<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	5            10	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	3	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	U/L				
<b>PRECISION:</b>	1				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	MAG	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Magnésio	<b>SAMPLE VOLUME:</b>	4 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	578	<b>CONCENTRATION:</b>	0	6.1	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	---		
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	-1      10	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	0.01				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
TEST NAME:	PROT	R1:	200 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	Proteína	SAMPLE VOLUME:	4 µL		
STANDARD N°:	(*)	R1 BLANK:	---		
ANALY. TYPE:	ENDPOINT	MIXED REAG BLK:	---		
PRI. WAVE:	546	CONCENTRATION:	0.2	15.0	
SEC. WAVE:	670	LINEARITY LIMIT:	---		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	0 33	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	g/dL				
PRECISION:	0.01				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	ONE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:	0				
CONCENTRATION:	#				

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**PARAMETERS**

<b>TEST NAME:</b>	PROTUE	<b>R1:</b>	200 µL
<b>TEST N°:</b>		<b>R2:</b>	0 µL
<b>FULL NAME:</b>	Proteinúria	<b>SAMPLE VOLUME:</b>	4 µL
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---
<b>ANALY. TYPE:</b>	ENDPOINT	<b>MIXED REAG BLK:</b>	---
<b>PRI. WAVE:</b>	578	<b>CONCENTRATION:</b>	8      400
<b>SEC. WAVE:</b>	---	<b>LINEARITY LIMIT:</b>	---
<b>TREND:</b>	ASCENDING	<b>SUBSTRATE LIMIT:</b>	---
<b>REAC. TIME:</b>	-1      17	<b>FACTOR:</b>	---
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---
<b>UNIT:</b>	mg/dL		
<b>PRECISION:</b>	2		

**REFERENCE**

N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)

**CALIBRATION RULE**

<b>RULE:</b>	ONE POINT LINEAR
<b>SENSITIVITY:</b>	
<b>REPLICATES:</b>	2
<b>INTERVAL:</b>	
<b>DIFFERENCE LIMIT:</b>	
<b>SD:</b>	0
<b>BLANK RESPONSE:</b>	
<b>ERROR LIMIT:</b>	
<b>COEFFICIENT:</b>	0
<b>CONCENTRATION:</b>	(#)

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	TGO	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	TGO	<b>SAMPLE VOLUME:</b>	20 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINETIC	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	340	<b>CONCENTRATION:</b>	4	600	
<b>SEC. WAVE:</b>	405	<b>LINEARITY LIMIT:</b>	0.2000		
<b>TREND:</b>	DESCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	3            13	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	U/L				
<b>PRECISION:</b>	1				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	TGP	<b>R1:</b>	200 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	TGP	<b>SAMPLE VOLUME:</b>	20 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINETIC	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	340	<b>CONCENTRATION:</b>	4	450	
<b>SEC. WAVE:</b>	405	<b>LINEARITY LIMIT:</b>	0.2000		
<b>TREND:</b>	DESCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	3            13	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	U/L				
<b>PRECISION:</b>	1				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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<u>PARAMETERS</u>					
TEST NAME:	TRI	R1:	300 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	Triglicérides	SAMPLE VOLUME:	3 µL		
STANDARD N°:	(*)	R1 BLANK:	---		
ANALY. TYPE:	ENDPOINT	MIXED REAG BLK:	---		
PRI. WAVE:	510	CONCENTRATION:	0	600	
SEC. WAVE:	---	LINEARITY LIMIT:	---		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	0 33	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	mg/dL				
PRECISION:	1				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	ONE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:	0				
CONCENTRATION:	#				

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<u>PARAMETERS</u>					
<b>TEST NAME:</b>	URE	<b>R1:</b>	300 µL		
<b>TEST N°:</b>		<b>R2:</b>	0 µL		
<b>FULL NAME:</b>	Ureia	<b>SAMPLE VOLUME:</b>	3 µL		
<b>STANDARD N°:</b>	(*)	<b>R1 BLANK:</b>	---		
<b>ANALY. TYPE:</b>	KINETIC	<b>MIXED REAG BLK:</b>	---		
<b>PRI. WAVE:</b>	340	<b>CONCENTRATION:</b>	0	230	
<b>SEC. WAVE:</b>	670	<b>LINEARITY LIMIT:</b>	0.2000		
<b>TREND:</b>	DESCENDING	<b>SUBSTRATE LIMIT:</b>	---		
<b>REAC. TIME:</b>	2	<b>FACTOR:</b>	---		
<b>INCUBA TIME:</b>	---	<b>PROZONE CHECK:</b>	---		
<b>UNIT:</b>	mg/dL				
<b>PRECISION:</b>	INTEGER				
<u>REFERENCE</u>					
<b>N°</b>	<b>GENDER</b>	<b>SAMPLE TYPE</b>	<b>AGE</b>	<b>LOW LIM</b>	<b>UP LIM</b>
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
<b>RULE:</b>	ONE POINT LINEAR				
<b>SENSITIVITY:</b>					
<b>REPLICATES:</b>	2				
<b>INTERVAL:</b>					
<b>DIFFERENCE LIMIT:</b>					
<b>SD:</b>	0				
<b>BLANK RESPONSE:</b>					
<b>ERROR LIMIT:</b>					
<b>COEFFICIENT:</b>	0				
<b>CONCENTRATION:</b>	#				

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