TURB ASO – ANTI-ESTREPTOLISINA O

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<u>PARAMETERS</u>					
TEST NAME:	ASO	R1:	240 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	ASO	SAMPLE VOLUME:	3 µL		
STANDARD N°:	(*)	R1 BLANK:	---		
ANALY. TYPE:	ENDPOINT	MIXED REAG BLK:	---		
PRI. WAVE:	546	CONCENTRATION:	3	800	
SEC. WAVE:	---	LINEARITY LIMIT:	---		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	-1 10	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	UI/mL				
PRECISION:	0.01				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	SINGLE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:					
CONCENTRATION:	#				

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<u>PARAMETERS</u>					
TEST NAME:	PCR Ultra	R1:	225 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	PCR Ultra	SAMPLE VOLUME:	3 µL		
STANDARD N°:	(*)	R1 BLANK:	---		
ANALY. TYPE:	ENDPOINT	MIXED REAG BLK:	---		
PRI. WAVE:	546	CONCENTRATION:	0	15	
SEC. WAVE:	---	LINEARITY LIMIT:	---		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	-1 18	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	mg/L				
PRECISION:	0.01				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	SINGLE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:					
CONCENTRATION:	#				

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<u>PARAMETERS</u>					
TEST NAME:	PCR	R1:	300 µL		
TEST N°:		R2:	0 µL		
FULL NAME:	PCR	SAMPLE VOLUME:	3 µL		
STANDARD N°:	(*)	R1 BLANK:	---		
ANALY. TYPE:	ENDPOINT	MIXED REAG BLK:	---		
PRI. WAVE:	546	CONCENTRATION:	1	100	
SEC. WAVE:	---	LINEARITY LIMIT:	---		
TREND:	ASCENDING	SUBSTRATE LIMIT:	---		
REAC. TIME:	-1 10	FACTOR:	---		
INCUBA TIME:	---	PROZONE CHECK:	---		
UNIT:	mg/L				
PRECISION:	0.01				
<u>REFERENCE</u>					
N°	GENDER	SAMPLE TYPE	AGE	LOW LIM	UP LIM
(*)	(*)	(*)	(*)	(*)	(*)
<u>CALIBRATION RULE</u>					
RULE:	SINGLE POINT LINEAR				
SENSITIVITY:					
REPLICATES:	2				
INTERVAL:					
DIFFERENCE LIMIT:					
SD:	0				
BLANK RESPONSE:					
ERROR LIMIT:					
COEFFICIENT:					
CONCENTRATION:	#				

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